

# Changing an R<sup>2</sup> CURTAIN SYSTEM



1 Open new curtain packaging at perforation.



2 Line up new curtain with curtain rod assembly and insert firmly.



3 Remove loose key from package.



4 Remove and discard both keys.



5 Gather old curtain and insert key through upper holes.



6 Remove paper band & plastic sleeve from new curtain.



7 Turn key toward you to release curtain.



8 Open curtain.



9 Discard old curtain.



10 No step ladders, no special training. Anyone can do it.

Web: [ICPmedical.com](http://ICPmedical.com) · 800-405-3044

Proud Supporter of

