

Bootie Butler EMLG | EMLG Electric Automatic Shoe Cover Dispenser

The Electric Butler (EMLG) is designed for moderate to high-volume areas where safety, speed, and compliance are critical. Its unique, sturdy design allows workers to put on shoe covers quickly, easily, and safely and the LED display shows when you need to replace the Shoe Cover Box.

EMLG Electric Automatic Shoe Cover Dispenser

- Easy to use: LED smart guide with LED light guides wearer exactly where to place their foot; light will turn off when the shoe cover is correctly fitted
- Easy-to-load Shoe Cover Box slides into the dispenser and LED light automatically shows quantity of shoe covers remaining in the box
- High-volume capacity (holds up to 100 pairs of shoe covers)
- Hands-Free operation
- Smart safety feature detects when foot is removed and reloads new shoe cover
- Reduces application time by workers/employees
- Increases worker safety and compliance
- Reduces contamination and errors



Loading and Using the Dispenser

1. Open Lid of the Shoe Cover Dispenser
2. Load the box with the arrow pointed towards the front. The number of shoe covers will appear on the screen.
3. Shut the cover of the shoe cover dispenser. The shoe cover dispenser will release one shoe cover automatically.
4. Place one foot into the dispenser. The dispenser will automatically fit the shoe cover to the wearer.
5. Remove foot and a new shoe cover will automatically dispense.

